


Directions for the new TASH phone MTI file for Vantage and Vanguard

This MTI file for your Vantage or Vanguard contains the infrared commands for the latest phone from TASH. In the PRC device toolbox IR set up menu, this phone is named Quick Phone.

1. Download the memory transfer file into your computer, and then save on a USB drive.
2. Put the USB drive into your Vantage or Vanguard.
3. Back up your custom user area onto the USB drive. This is just a pre-caution, and good practice for every time you are trying something new.
4. Transfer the new file that you downloaded from PRC's web site. Put this new file into your custom user area in the Vantage or Vanguard.
5. In the Transfer Memory menu, when you see the flag that asks if you want to erase all vocabulary in the area, select NO.

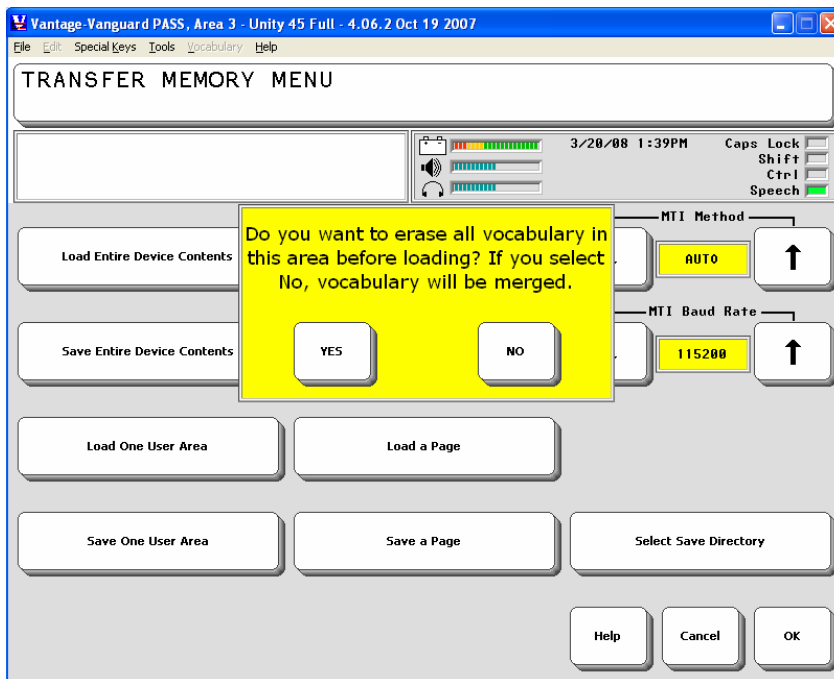

6. When the transfer is done, you will have the Quick Phone IR commands in the IR set-up menu, and a 45 location page for the Quick Phone in your custom user area. You will need to link that page somewhere in your custom area. If you want to expand the page to more locations, use the Choose Overlay menu in the device Toolbox.