

Importing Pictures from a Scanner into the Vanguard, Vantage, Pathfinder, or SpringBoard

Step 1: Importing a picture from the scanner to a computer and saving it in a “paint” program.

1. Go to scanning software on the computer.
2. When it asks what program to scan the picture into, say “Paint” and click Scan
3. Crop the picture before you Accept it so you can just have the area that you want on the device. You should use the selector tool to crop the picture.
4. Then click Accept.
5. When you have the picture open in the “Paint” program:
6. Go to Save As under the File menu
7. Save in 256 color bmp format
8. Name the picture
9. Save in My Documents folder
10. Now you can choose the “Paint” program and the scanning program unless you want to scan any other pictures.

Step 2: Using Memory Transfer to import this into the Vanguard or Pathfinder.

1. Go to Transfer Memory Menu on the device with the device hooked up to the computer in the Serial Port (Modem).
2. Open the memory transfer program “Mti exe.” on the computer.
3. Press Import Icons on the device.
4. When My Documents comes open, click on the picture that will be on the device. (Note: try with or without Maintain Aspect Ratio checked.)
5. Click OK and the picture will be imported. Then click OK again when it stops.
6. At this point you can add another picture if you want by going back through steps D through G or you can quit the “Mti exe.” program.
7. When you edit page that you want to put the picture on, press Change Icon. You will find your picture under the IMPORTS category.