

QRM2 with Speed Handle



The Speed Handle allows you to easily make adjustments to the QRM2.



Slide the QRM2 onto the mount on the back of your device. The round black knob should be facing towards the top of your device. Lift the black knob up and finish sliding the QRM2 onto the mount. Release the knob. It should drop into small hole. This latches the QRM2 onto the mount.

To remove the QRM2 from your device, pull up on the round knob and slide the QRM2 down and off



Now place your device on your table stand or wheelchair mount. The handle on the QRM2 should be pointing down towards the bottom of your device. Turn the QRM2 handle counter-clockwise to loosen it and make any adjustments. Then tighten the handle by turning it clockwise.

To make small adjustments, press and hold the small gray button on the handle while turning the handle. When you release the button, the handle will lock in position.

If you would rather have something to adjust that does not stick out, you can replace the handle with the screw that came in the bag with the QRM2. There is also an L-shaped wrench in the bag.



Remove the handle by simply turning it counter-clockwise until it is loose enough to pull out.



Remove the small washer from the handle.



Place the washer on the screw that came in the bag with the QRM. Place the screw with washer on it into the hole in the QRM.



Use the small silver L-shaped wrench to tighten the screw.

From now on, use this L-shaped wrench to loosen and tighten the screw to make adjustments to the QRM2.