


# Accent™<sub>700</sub>

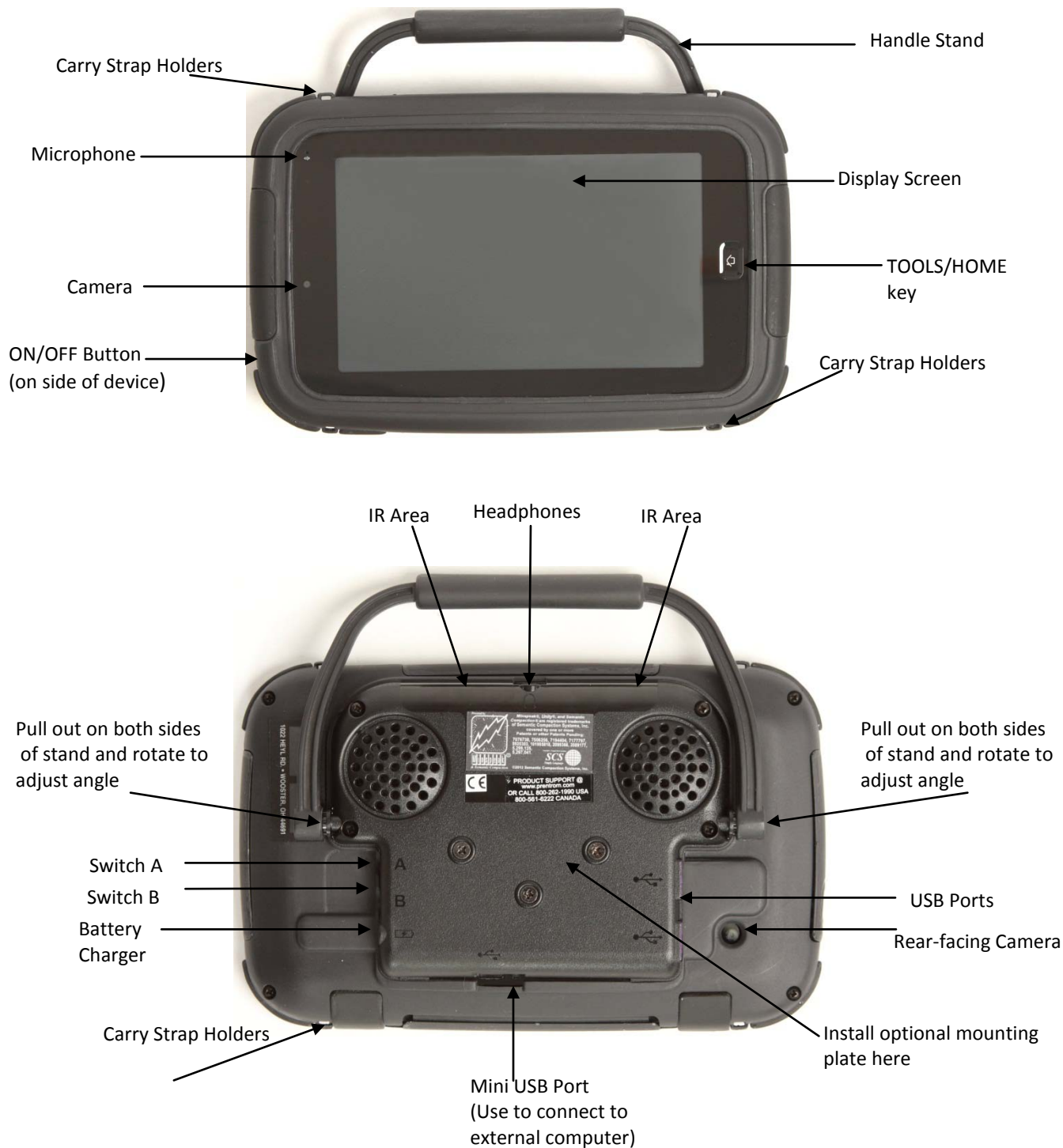

## Add a Screen Protector

A Screen Protector Kit is an option you can purchase. Three screen protectors have a matte finish and three have a lustre finish. The matte finish offers the most glare resistance. Matte protectors can be identified by the small red square on the back of their protective holders.

- **Lightly mist** your device display screen with water.
- Peel a screen protector from its protective holder. The side of the protector that was against the holder should be placed against the display.
- Slide the protector around until it covers the display from corner to corner.



- Use the squeegee to smooth out the protector. Remove air bubbles and wrinkles.
- Dry the protector and inside edges of the case with a lint-free cloth.

## Add a Touchguide

Touchguides are an option you can purchase. You can use a touchguide with or without a screen protector.



Insert the bottom tabs of the touchguide into the slots at the bottom edge of the frame.



Bend the frame backwards. Insert the upper tabs behind the top of the frame. The touchguide will take some effort to bend.

## Remove the Touchguide



Grasp the **thicker piece** of plastic above the first full row of keyholes.

Pull out and down on the touchguide until the upper tabs pop out. **Do not pull on the thin pieces of plastic!**

## Add the Mounting Plate

The mounting plate is an option you can purchase.



Remove the three screws on back of device. Put them in a safe place.



Place mounting plate over holes. **Attach it using the screws that came with the mounting plate.**

## Wireless Internet Connection




You must have access to a local Wi-Fi network.

### Setting Up a Wireless Internet Connection

⚠ **IMPORTANT NOTE:** PRC is not responsible for the set-up of your wireless network.

⚠ If you have, and can use, a USB mouse, you can plug it into your device and use it to select icons on your desktop.

- Turn your device **On**.

- Select the **HOME/TOOLS** key  on the right side of the display screen or select the  **TOOLS** key to the right of the Text Display Area.
- Select the **TOOLBOX** key from the **TOOLS** menu.
- Select the **MAINTENANCE MENU** key from the **Toolbox**.
- Select the **Hardware Diagnostics** key.
- Select the **Network Setup** key.
- On the screen, select the **Connect or disconnect** link.
- You see a list of available networks to connect to and their signal strength.
- Select the network you want to connect to.
- Tap **Connect**.
- Follow the directions on your display. You may have to enter a network key or password.
- When you are connected, the Wi-Fi icon in the task bar changes to bright white.
- Exit the menu and go to the Internet to test the connection. The Internet Explorer icon  is on your Windows display screen.



⚠ Most MAPS also have the Internet Explorer icon on the Computer Pages.

## PRC Bluetooth Wireless Adapter

➡Bluetooth capability is automatically enabled in the Accent™ 700. It is an option you can purchase with the Accent™ 700-SB.

You can only use one Bluetooth device at a time with your device. If you are using a Bluetooth switch and want to connect to an external computer, you must unpair your switch and pair with the Wireless Adapter instead.

1. Plug the Bluetooth wireless adapter into the USB port on your computer. Remember where the white circle is on the adapter. The LED on the adapter may flash for a moment when you plug it in.
2. Make sure your device is On.

3. Select the **HOME/TOOLS** key  on the right side of the display screen or select the  key to the right of the Text Display Area.

4. Select the **TOOLBOX** key .

5. In the Toolbox, open the **OUTPUT MENU**.
6. Set **Output** to **ON**.
7. Set **Output Destination** to **EXTERNAL**.
8. Set **Output Method** to **Bluetooth**.
9. Press the **Pair with BT Adapter** option.
10. You will see a yellow box telling you to **press the Pair button on the adapter**. Press the **white circle** label on the Bluetooth adapter.
11. The LED on the adapter will begin to blink. When you see a solid light, the adapter has paired with your device and computer. This can take up to 20-30 seconds. In the Status Box in the **OUTPUT MENU** you will see "Paired, Connected".
12. Exit the **OUTPUT MENU** and go to your spelling keyboard.
13. Open a word document or an application on your computer that you can type into.
14. Begin typing on your spelling keyboard. Your information should appear on your computer screen.

➡The Bluetooth adapter has a wide range. Your communication device and the adapter do not have to "see" each other to work.

## Cleaning Your Device

**To clean the case**, turn the device OFF. Wipe the case with a damp, lint-free cloth. Damp means wrung out, never dripping. Dry the case thoroughly before turning it ON.

**To clean the display**, turn the device OFF. Use a household window cleaner (e.g., Windex®) or a cleaner designated for a computer or flat screen TV or use rubbing alcohol. Lightly spray the screen. Dry thoroughly and polish with a lint-free cloth.

**Touchguides** can be washed in hot, soapy water. Dry thoroughly.

**To disinfect the case**: Use a solution of vinegar and water (1/4 cup vinegar to 1 cup water). Wipe the case (not the display) with a cloth dampened (wrung out, not dripping) in this mixture then let the case air dry. The **touchguide** can be washed in vinegar and water and left to air dry. **NEVER USE VINEGAR OR A VINEGAR/WATER SOLUTION ON THE DISPLAY!**

➡ For more device and troubleshooting information download the **Hardware Manual for the Accent 700**: [www.prentrom.com](http://www.prentrom.com). Click on **Support, Device and Accessory Manuals**, select your **device**, select the download you want.