

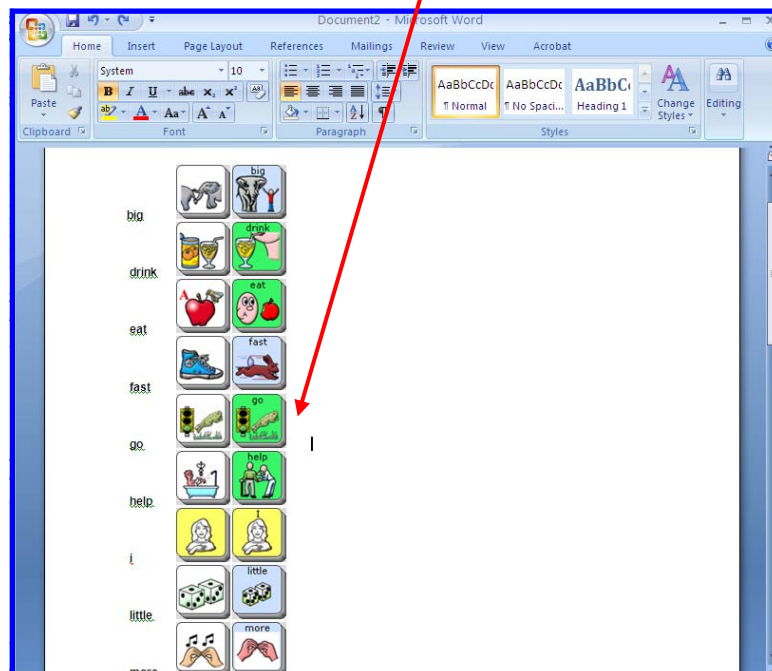


PASS-Create Smart Chart Instructions

A “Smart Chart” is a cheat sheet based on a set of words created using the Vocabulary Builder feature in ECO, Vantage Lite, and Vanguard devices. **Vocabulary Builder** is a therapeutic tool that allows you to easily teach new vocabulary within sequenced Unity. Using the PASS emulation software on your computer, available from the PRC website, you can create support materials to accompany Vocabulary Builder and assist with teaching vocabulary in the device.

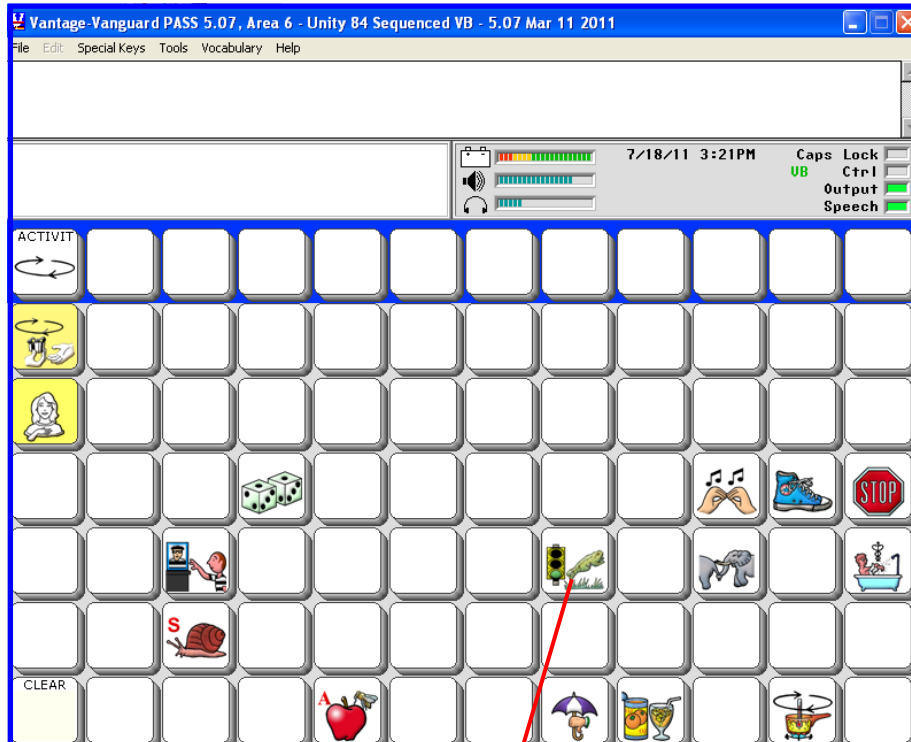



1. In PASS, use **File Menu** and **Preferences** to select “**PRC style**” under Vocabulary Builder Style options. Select **OK** when finished.

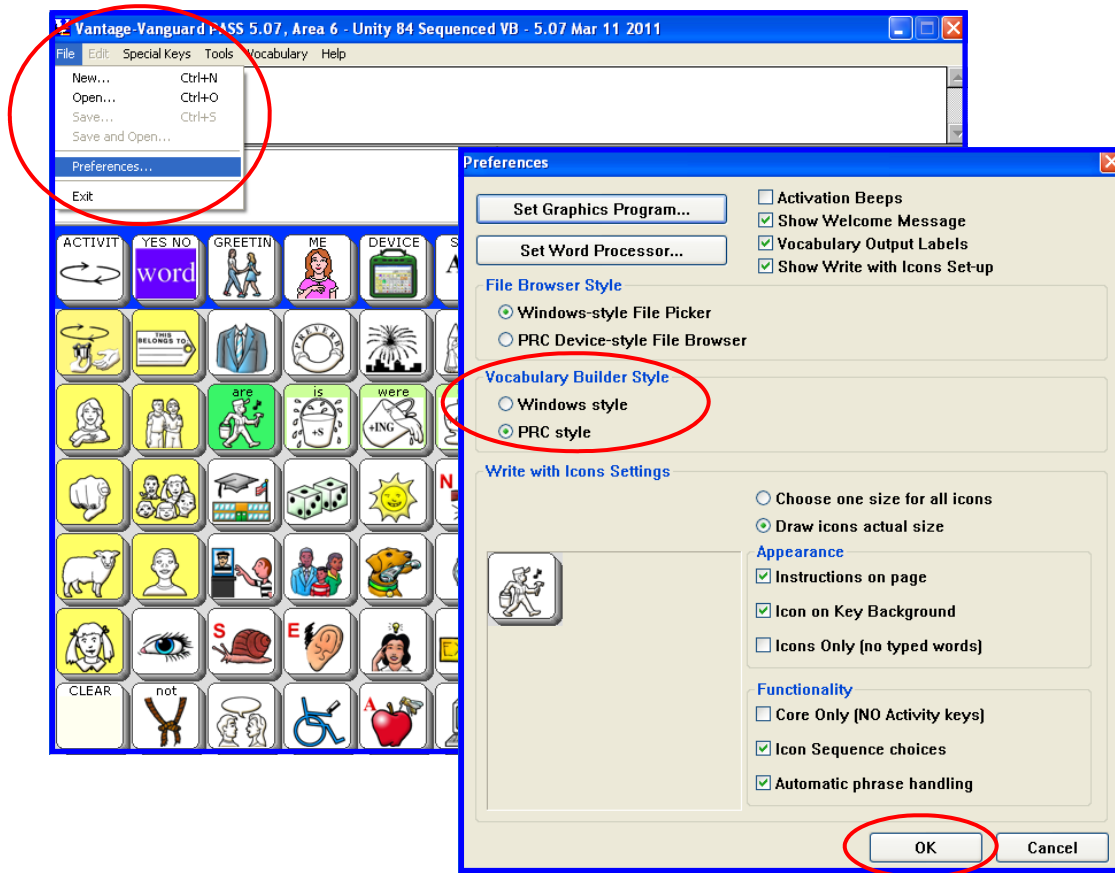


2. Use Vocabulary Builder to create the set of words you wish to teach. Refer to the Vocabulary Builder Manual and Vocabulary Builder Cheat Sheet for instructions on how to use the Vocabulary Builder tool: http://www.prentrom.com/training/materials#course_7

For additional help with the Vocabulary Builder tool, contact your Regional Consultant for training or register for the online Vocabulary Builder e-Training at: <http://www.prentrom.com/training>.


3. With a Vocabulary Builder set loaded in the PASS software, select **Vocabulary>Vocabulary Display Options>Create Smart Chart**.

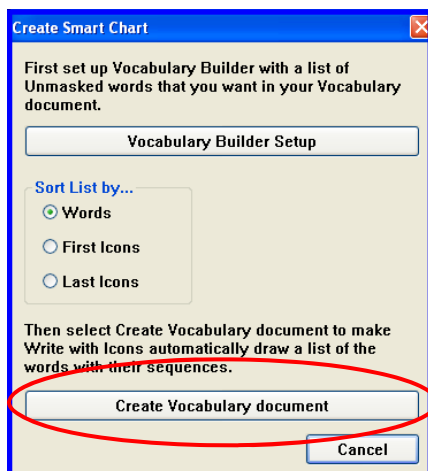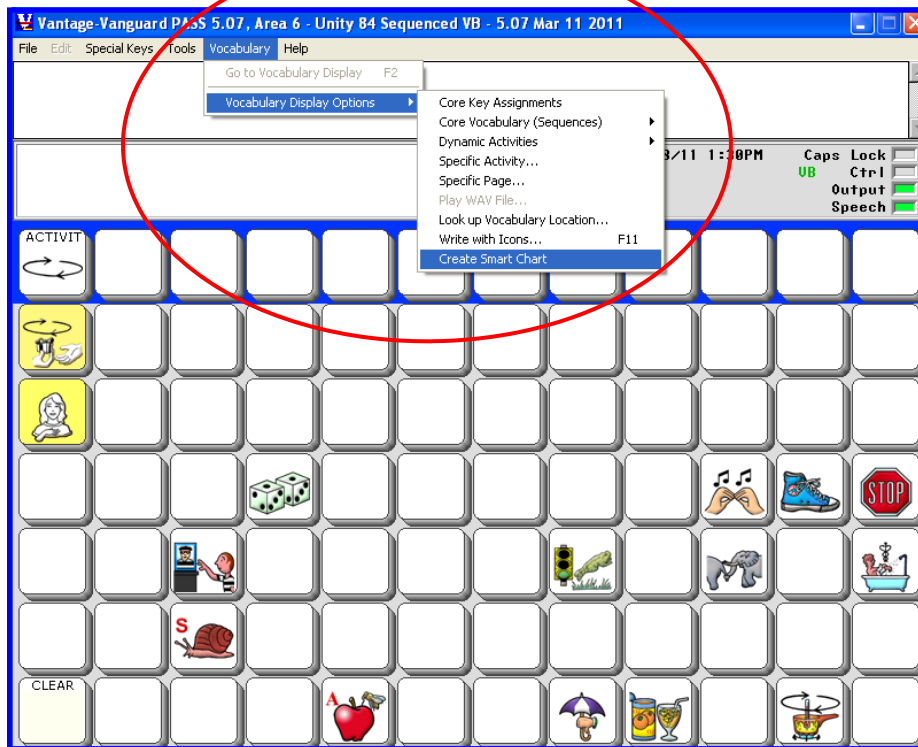



4. A Smart Chart is created!

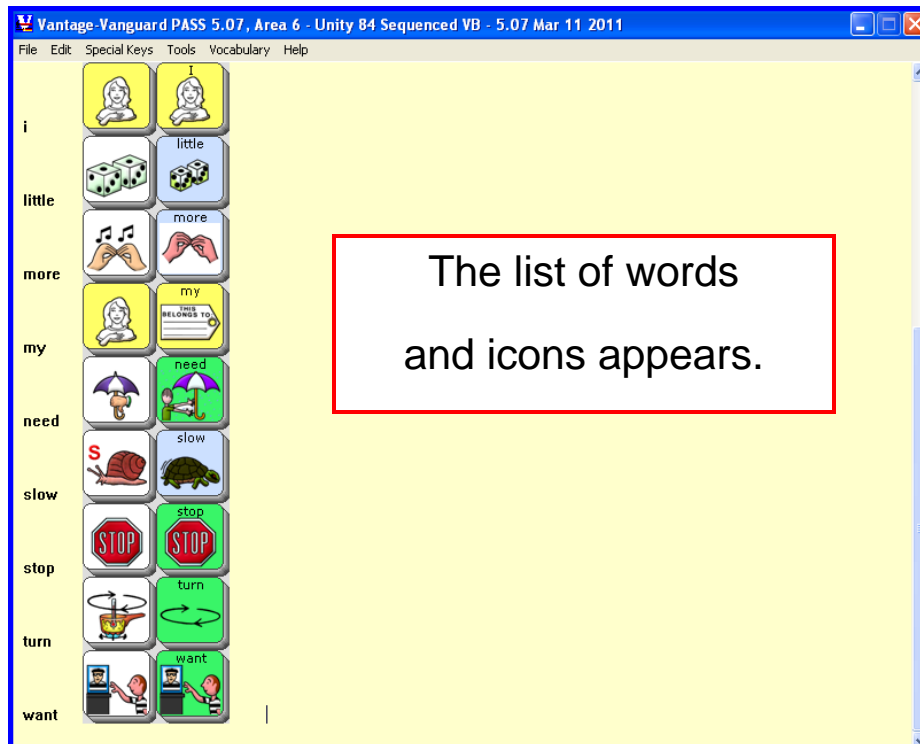


5. Select ALL words and icons.

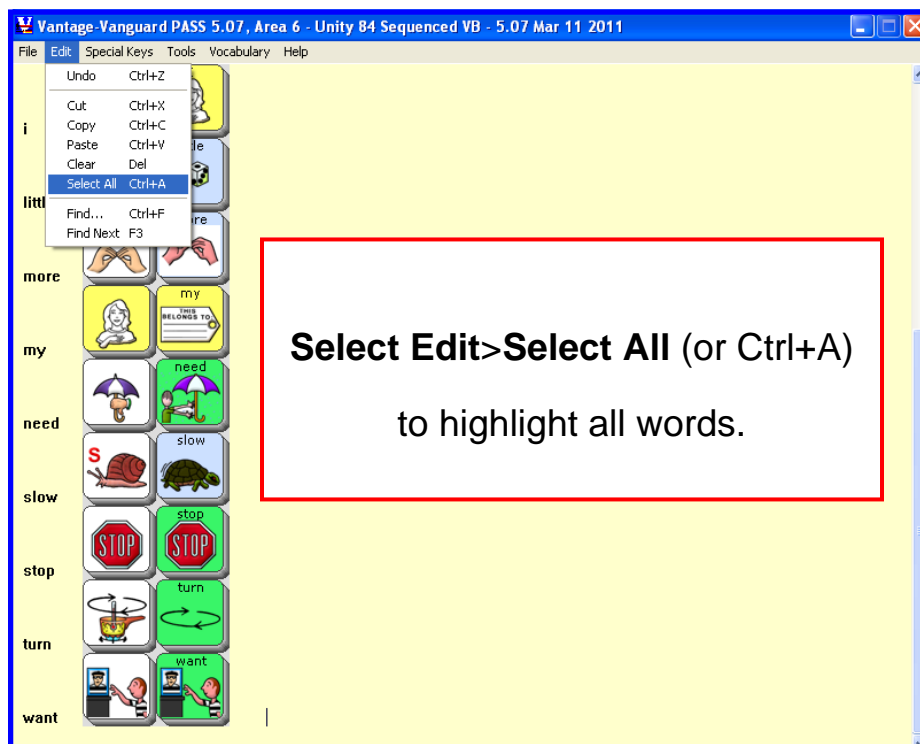

6. Copy and Paste into Microsoft Word.

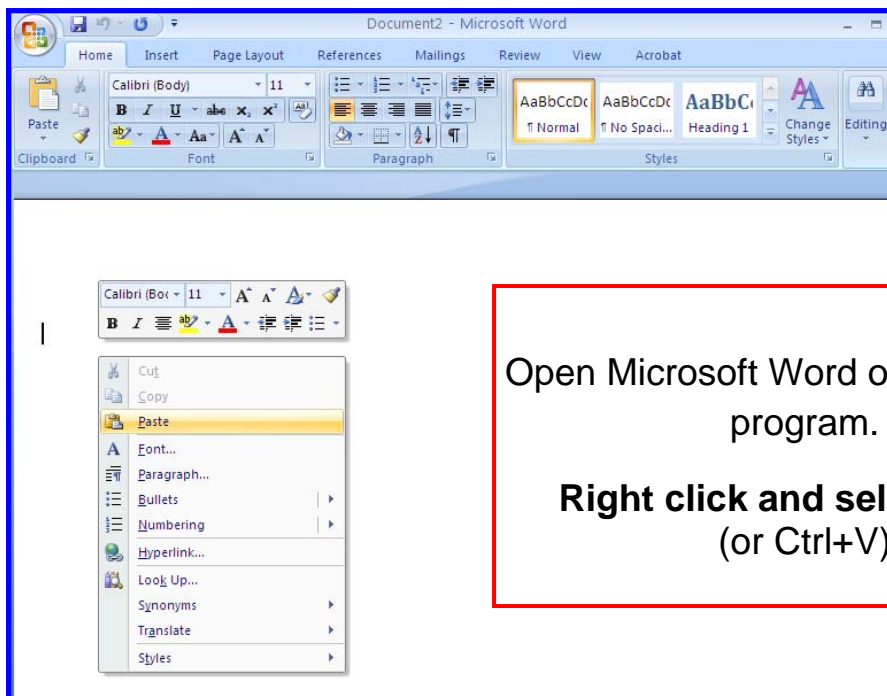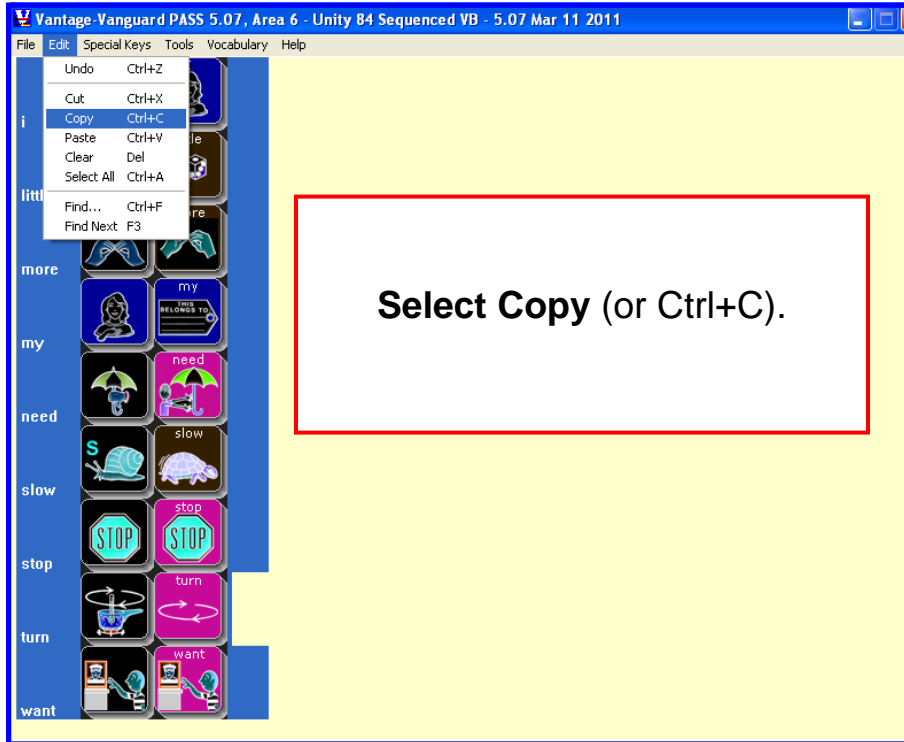

7. Word/icon list can be manipulated as needed in Microsoft Word or other program.

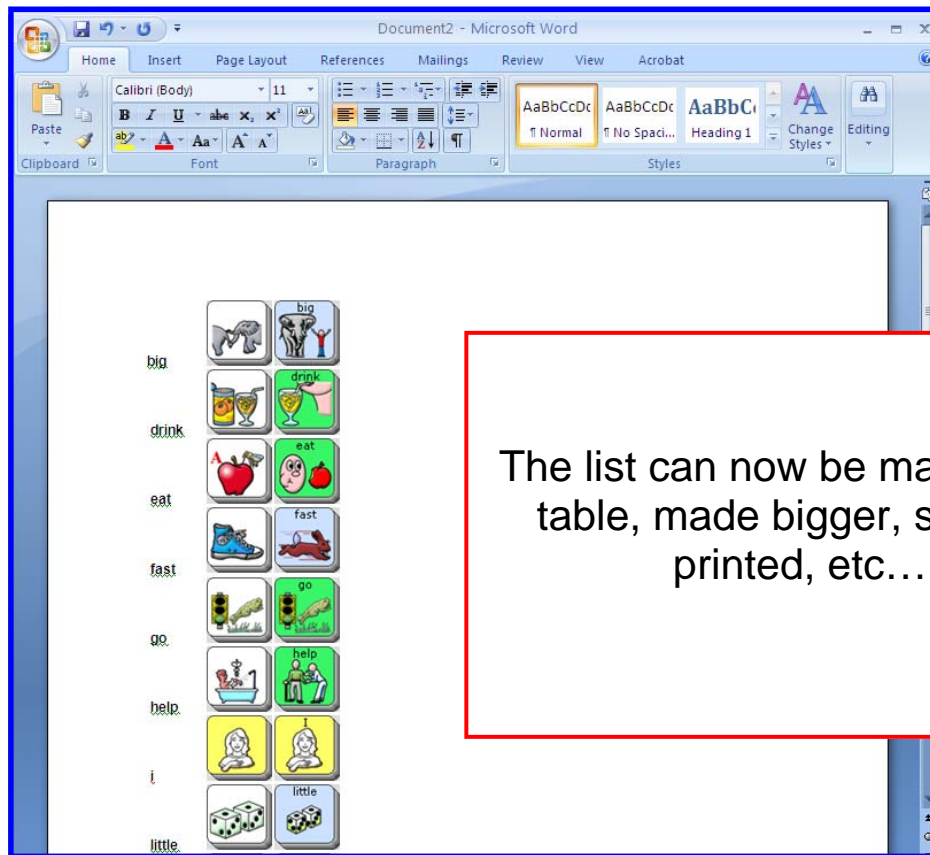

8. Use the printed copy of words and icons to help teach vocabulary on a device!!