

# SPRINGBOARD LITE – AAC MADE SIMPLE



Available in 4 Bold Colors!

Midnight Black (BLK)

Seafoam Green (SEA)

Sunset Orange (ORG)

Vineyard Grape (GRP)


No longer available




# Exploring the Hardware



# Top of SpringBoard Lite

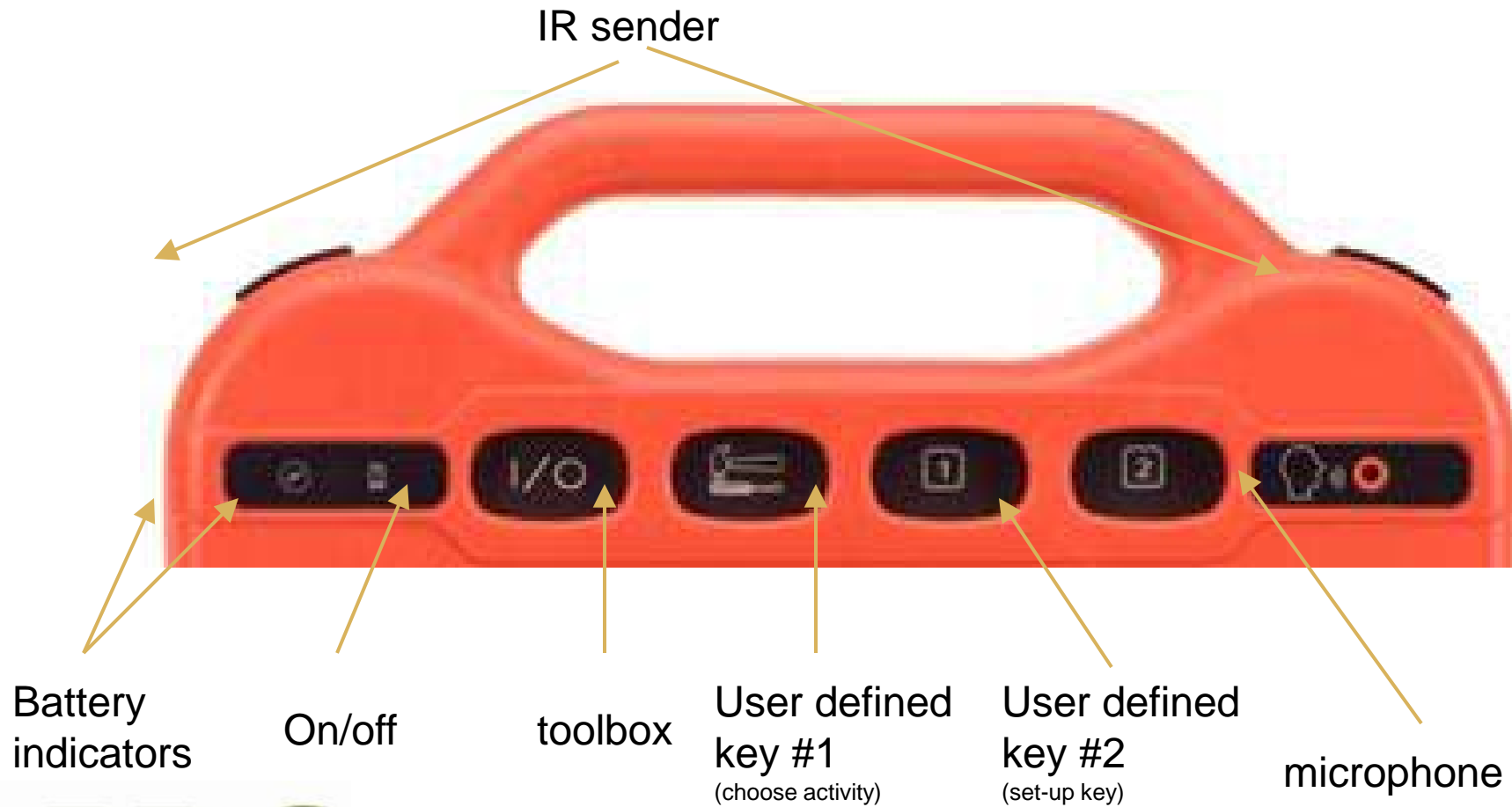

# Right Side of SpringBoard Lite

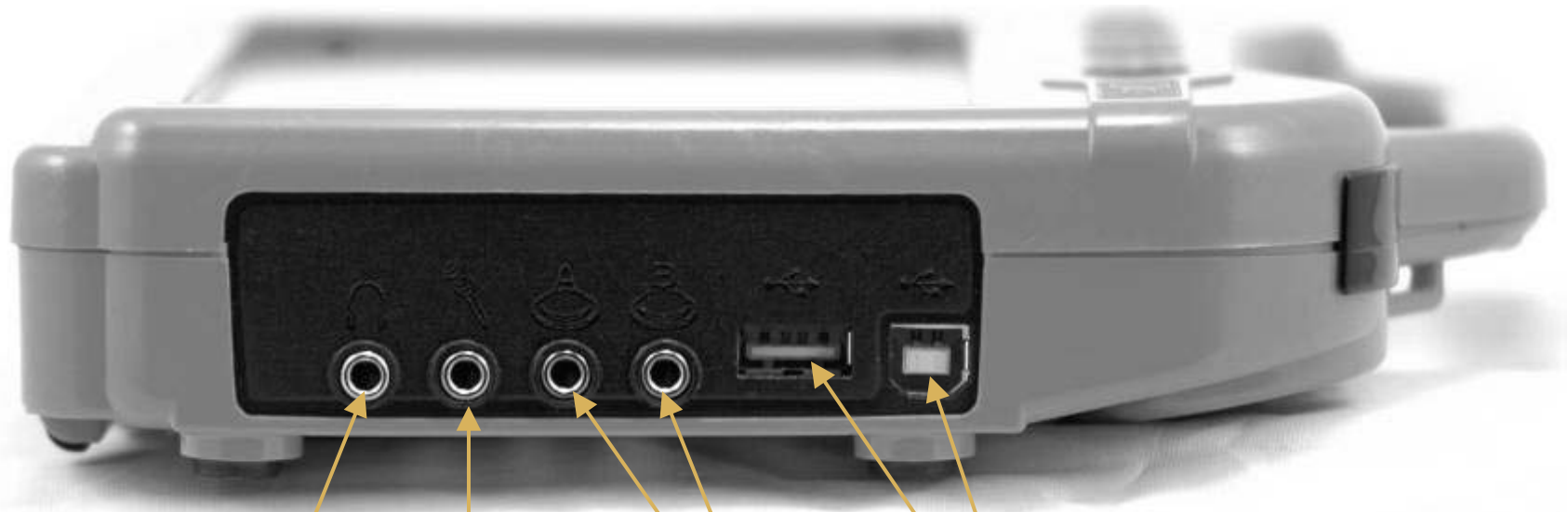

Headphone  
jack

Microphone  
jack

Switch  
ports

USB ports



PRC

# Left Side of SpringBoard Lite



SD Card slot  
(used for MP3's)

Charger  
port




# Back of SpringBoard Lite

Reset

Place to add straps

Battery Door

Screws for battery and memory card

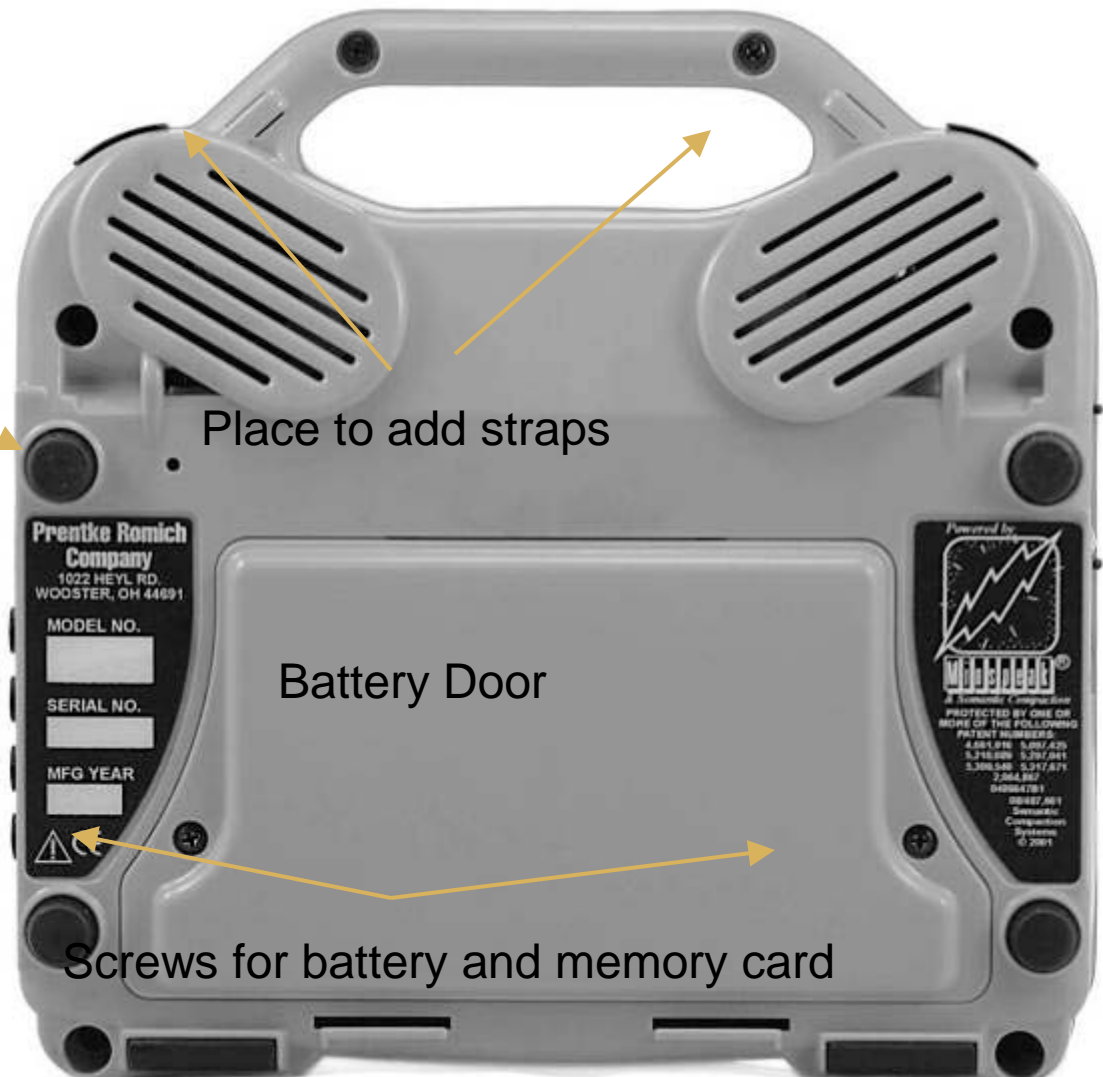

# Exploring the vocabulary





# Core Vocabulary

- Core vocabulary includes a **small** set of 400-500 words that make up **80%** of what we say everyday
  - Same words across gender, age, topic, setting, disability, profession, etc.
  - Include pronouns, verbs, adjectives, adverbs, question words, demonstratives, prepositions
  - Not very good picture-producers
    - ✦ truck vs. that
    - ✦ play vs. toaster

To communicate effectively, you must have access to core vocabulary!
































# Fringe Vocabulary

- Fringe vocabulary is a large set of thousands of words that make up 20% of what we say everyday
  - ▣ Different words for different people depending on gender, age, topic, setting, disability, profession, etc.
  - ▣ Mostly nouns – more specific than core vocabulary
  - ▣ Easy to depict on a picture symbol

In PRC devices, fringe vocabulary is found in the Activity Row or on Pages.



The Activity Row and Pages link include fringe vocabulary. Spend 20% of your time teaching these words.

YES NO word	ME 					GREETINGS 	FUN 
NAMES NAME	my THIS BELONGS TO:	is 	please 	that 	to	what word	more 
PAGES 		play 	like 	work 	feel 	read 	stop STOP
ACTIVITY 	you 	want 	come 	do 	go 	get 	help 
CLEAR	it 	don't 	eat 	make 	drink 	turn 	color 

80% of the display is dedicated to core vocabulary. Spend 80% of your time teaching these words.










# Samples of Pages










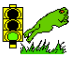

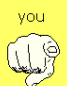

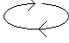

		<b>Categories</b>			TO CORE →
	ANIMALS 	CLASSES 		CLOTHING 	JOBS 
PAGES 		OPPOSITES 		PREP 	SHAPES 
	BODY 		SPORTS 		TOYS 
CLEAR					








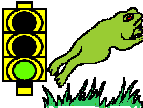
	yes 	<b>Weather</b>				no 	TO CORE →
weather 		is 	hot 	cold 	and 		
PAGES 	sunny 	cloudy 	rainy 	windy 	foggy 	snowy 	icy 
DELETE 		CALENDAR 		frosty 	stormy 	thunder 	lightning 
CLEAR	It 			hurricane 	tornado 	earthquake 	




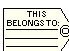

















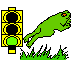







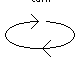

	yes 	<b>Sensory</b>				no 	TO CORE →
					swing 	ball 	trampoline 
PAGES 	I 			treadmill 	tunnel 	stander 	skateboard 
		want 		cushion 	weighted blanket 	ball pit 	beanbag 
CLEAR		don't 		koosh ball 	bands 	fan 	light 

# User Areas

yes 			CLEAR	no 
sing 	play 			
eat 	read 			

YES NO <b>word</b>	PAGES 		ME 	GREETINGS 
my 	that 	what ? word	more 	stop 
CHOOSE ACTIVITY	I 	want 	go 	help 
CLEAR	you 	don't 	turn 	color 

Hi 	PAGES 		CLEAR	Good-bye 
I 	want 	more 	stop 	
you 	don't 	go 	help 	

YES NO <b>word</b>	ME 		GREETINGS 	FLIN 			
NAMES <b>NAME</b>	my 	is 	please 	that 	what ? word	more 	
PAGES 	I 	play 	like 	work 	feel 	read 	stop 
ACTIVITY 	you 	want 	come 	do 	go 	get 	help 
CLEAR	it 	don't 	eat 	make 	drink 	turn 	color 



# To find your local consultant...



[Home](#) | [My Account](#) | [Subscribe](#) | [Send to Friend](#) | [Contact](#)

Search

[PRODUCTS](#) ■ [FUNDING](#) ■ [TRAINING](#) ■ [SUPPORT](#) ■ [EDUCATIONAL RESOURCES](#) ■ [ABOUT US](#) ■ [NEWS AND SUCCESS STORIES](#)

## Quick Start

[What is AAC?](#)

[About the Unity Language](#)

[Why Choose PRC?](#)

[Success St](#)

[AAC Language](#)



## Our Devices Have People Talking

Read Kim Vuong's AAC Success Story

### FIND YOUR CONSULTANT

### FIND INFO FOR YOU

### NEW SOFTWARE UPDATES!

### LATEST NEWS

Find

Enter your ZIP to locate a PRC consultant in your area.

[Adult](#)

[Child](#)

[Caregiver](#)

[Vantage Lite \(Version 5.08\)](#)

[ECO2 \(Version 3.0\)](#)

[Celebrating 45 Years of Communication Success](#)

[We've Gone Digital!](#)

[PRC Named to Top Workplaces for Second Consecutive Year](#)



[www.prc.com](http://www.prc.com)



# UNITY<sup>®</sup> Resources!

## AAC Language Lab

<http://www.aaclanguagelab.com>



<http://www.aacandautism.com>




# Thank you!

Learn about new training opportunities and connect with other AAC users

16



<https://www.facebook.com/PrentkeRomichCompany>

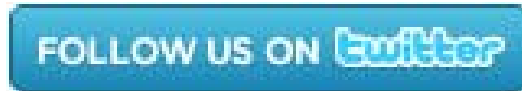

<http://twitter.com//PrentkeRomich>



<https://www.YouTube.com/PRCAccess>

